

## **SILVERLIFE TOPICAL SPRAYER COMPOSITION**

Silver ions 10mcg/ml, De-ionised water

## **INDICATION**

SilverLife Topical Sprayer is used in the cleaning and dressing of wounds.

SilverLife Topical Sprayer promotes faster healing in cuts, burns, bruises and wounds

## **DOSAGE**

Spray on affected area twice daily, or directed by physician

## **HOW TO APPLY SILVERLIFE TOPICAL SPRAYER**

Spray SilverLife directly on, and around the wound area.

Spray SilverLife on a sterile wound dressing material.

Spray SilverLife on a new sterile gauze swab, and place it on the wound area.

Spray SilverLife directly on wounds that require no dressing.

For fungal infections of the skin, clean the affected area with SilverLife.

## **BENEFITS OF USING SILVERLIFE TOPICAL SPRAYER**

1. Kills bacteria and fungus faster.
2. It's non - toxic
3. It's non - staining.
4. It stimulates tissue growth faster.
5. There is a minimum risk of maceration.
6. Eliminates bad odour.

## **CONTRAINDICATION**

Persons with known allergy to Silver

## **PRECAUTIONS.**

1. For external use only.
2. Close the bottle with the cap immediately after spraying to prevent contamination
3. Do not place the sprayer cap in direct contact with the wound. In case the sprayer cap is in direct contact with the wound, Discard the bottle and replace it with a new bottle.
4. If a new bottle has been tampered with, discard and get a new, untampered bottle.

## **STORAGE**

Store below 25°C.

**Shake well before use.**